

Lane Ram Ejaculator

For best results we suggest that the probe be clean and lightly lubricated. After insertion of the probe, massage the prostate with the probe 8 to 10 times. Push the button for 4 to 6 seconds, release for 5 to 8 seconds.

Massage during the rest period is recommended. The number of times and length of time will be determined by ejaculation of the ram.

New Feature

The Ram Ejaculator now has a High/Low switch on the back of the unit. The high setting is the normal ram setting that we have used for the last 25 years. The low setting works well for goats and some ram breeds. The switch in the low position cuts the power in half.

Care of the Lane Ram Ejaculator

Use Duracell 9 volt batteries when replacing batteries. The electrodes are made of brass and will corrode. Occasionally clean the electrodes with the scotch-brite pad that is included. Discoloration of the electrodes will affect the performance of the Ram Ejaculator.

For service or additional supplies please contact us at:

Lane Manufacturing, Inc
2045 S Valentia Street Unit 11
Denver CO 80231
Phone 800-777-2603
email: lanemfg@earthlink.net

09/06